




Practice Suture Kit

Use this Practice Suture Kit to practice wound closure and surgical incisions.

Guidelines



Always begin by washing your hands and using gloves.

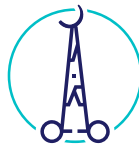

Use the needle holder to grab the needle and make sure it is locked in place. Pull the thread from the suture kit.



Use the forceps to handle the tissue.



Rotate your wrist to drive the curved needle through the skin. Twist so that the needle starts coming from the other side of the wound.



Once the needle has been taken through both sides of the wound, tie a knot.

Warning!

The Practice Suture Kit contains sharp objects, be extremely careful when handling. Surgical Tool Set to only be used on Practice Pad included in this case. Always wash your hands before using the kit to minimize the risk of infection if an accident occurs. Gloves are preferred. Use disinfectant or antiseptic solution to rinse the tools after each use.

What's in the Kit?



Adson/Toothed Forceps

To handle skin and stabilize the wound tissue.



Hegar Needle Holder/Driver

To firmly hold or secure the needle when suturing.



Surgical Scissors

To cut the suture material after knot.



Curved Mosquito Forceps

To halt blood flow so cauterization can be performed.



Scalpel Handle

To help to utilize the scalpel blade for making incisions.




4/0 Nylon Thread Sutures with 3/8 Needle

Used to suture.



Scalpel Blades

To attach to the scalpel handle for making incisions.

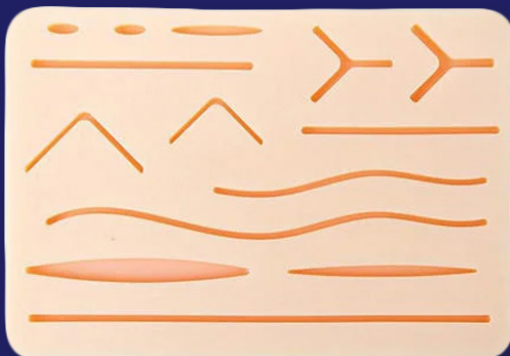

Practice Pad

Consists of 3 layers: skin, fat, and muscles to simulate real human flesh.



Carry-all Leather Kit

For storing both the Practice Pad and Surgical Tool Set.